


# GOT A LIGHT BUDDY?

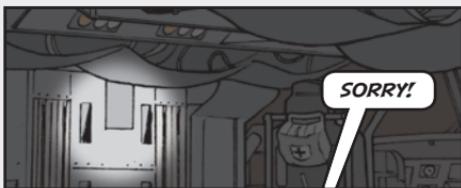


**T**ired of holding a mini-light in your teeth? If you've been looking for light bulbs or dome light assemblies for the interior in the medical Black Hawk, look no further.

For **light bulbs**, order NSN 6240-12-344-8188, which brings you a single halogen light bulb.

**Dome assemblies?** NSN 6220-12-381-0172 brings you the two-bulb dome light assembly found in the medical side panels.

NSN 6220-01-569-0543 gets you the **three-bulb dome assembly** used in the center overhead light bar and the left/right aft facing controls panels.

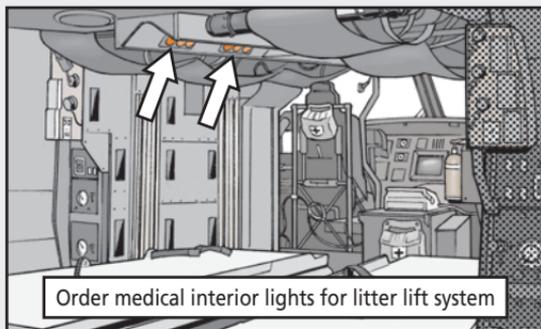

Want to decrease your workload? When replacing a halogen bulb, do not touch the bulb with your bare hands. The oils from your skin will damage the quartz surface when the light is turned on.

Always handle halogen bulbs with a clean towel or clean, lint-free cloth. If you accidentally touch a bulb with your bare hands, clean it with rubbing alcohol, NSN 6505-00-656-8366, and ensure it is dry it before installation. Keeping it clean will light up your world!